

EASY, ACCURATE RESULTS

# LEUKO EZ VUE®

*Only rapid test to detect intestinal inflammation from enteric infection*

- Detects elevated levels of fecal lactoferrin, a marker of fecal leukocytes and an indicator of intestinal inflammation
- Simpler and more accurate than microscopy for fecal leukocytes
- Rapid results in 10 minutes



LEUKO EZ VUE®

## Product Order Information

Product Name	Cat. No.	Sample Type	Time to Result	Format
LEUKO EZ VUE®	T5019	Fresh, frozen, refrigerated or room temperature	10 minutes	Lateral flow qualitative

## The LEUKO EZ VUE® test is simpler and more accurate

### LEUKO EZ VUE®

### Microscopy

- Standardized and reproducible results in 10 minutes
- Clear results are unaffected by cell breakdown
- Samples can be stored up to 2 weeks refrigerated or at room temperature without loss of sensitivity
- Can be batched or run individually for on-demand testing
- Subjective interpretation can vary by technician
- Must be run immediately to minimize cellular breakdown
- Tedious to perform and time consuming
- Significant technical training needed

## Assay Protocol

1 Add 2.5mL *Diluent* to a tube




2 Fill flared tip with sample and add to tube containing *Diluent*.



3 Add filter tip to tube. Vortex to mix.



4 Add 5 drops to *Sample Well*. Read at 10 minutes.

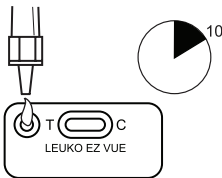

5

Sample Well      Reaction Window

LEUKO EZ VUE

Positive

LEUKO EZ VUE

Negative

Distributed by



Phone: +43 (0) 2244 296 31  
www.labraum.at | info@labraum.at

Developed and manufactured by



PROVEN DIAGNOSTIC PERFORMANCE

www.techlab.com

TECHLAB, Inc. is a leading developer and manufacturer of infectious disease diagnostics products worldwide. Since 1989, the company has offered a quality portfolio of diagnostic tests which now includes assays for *C. difficile*, *Campylobacter*, Shiga toxin producing *E. coli*, *H. pylori*, protozoan parasites, and biomarkers of intestinal inflammation.

